

# OPERATION CHART OF DISPOSABLE CIRCUMCISION STAPLER

**1** Use a measuring ruler to choose the right model of stapler.

**2** Screw off the adjusting nut counter clockwise and take out the glans cover.

**3** Lift the foreskin with hemostat and put the glans cover onto the glans.

**4** Make sure the edge of glans cover lies at the coronary sulcus and on the same level as coronary sulcus.

**5** Screw off the adjusting nut counter clockwise and take out the glans cover.

**7** Keep hold of the handle for 5-10 seconds

**6** When the end of pull rod is on the same level with the surface of metal insert inside the adjusting nut, the position of adjusting nut is right and the safety clip can be removed.

**5** Pull apart the nail cover, insert the pull rod into the instrument and screw the adjusting nut.

## APPLICATION SCOPE

It is applicable for clinical circumcision suturing operation

## PRODUCT FEATURES

1. The operation is simple, fast and convenient.
2. Circumcision and suture are finished at one time, which greatly shortens the operation time.
3. It can reduce hemorrhage during surgery and edema after surgery.
4. The suturing nails will fall off automatically and no more removal of suture will be required. The incision will be neater after healing.



The nail foot is single chamfer, which is easy for shaping and with less thrust.

Blood vessels can be effectively avoided through identification on the nail cartridge.

New design of annular cutting knife ensures complete incision of tissues to reduce damage to tissues and ensure neat incision.

Space between nails is small, which can reduce hemorrhage.

## ADVANTAGES COMPARED WITH OTHER SIMILAR PRODUCTS

1. The nail feet are much denser and the space between nail feet is 0.5mm, which can ensure no hemorrhage during and after surgery.
2. Nail-off after surgery is more effective.
3. The raw material is imported, of which the quality is better.

Model	Diameter(mm)	Nails
KYBW-12	12	10
KYBW-15	15	12
KYBW-19	19	15
KYBW-22	22	18
KYBW-26	26	20
KYBW-31	31	24
KYBW-36	36	27