

## Single Use Obstetric Suction Cup for instrumental extraction



HTKD-Med Suction Cup

The HTKD-Med suction cup is a versatile obstetric suction cup used for instrumental extraction procedures. It is suitable for all fetal presentations

VERSATILE  
EFFICIENT  
ECONOMIC

### Principle

The HTKD-Med obstetric suction cup is a simple and effective single-use device for instrumental extraction. The traction hose fixed in the center of the suction cup is housed in the tunnel provided for this purpose which allows easy insertion of the suction cup and optimal positioning on the flexion point for all fetal presentations.

Once the device is correctly positioned, the hand pump associated with the traction handle generates a vacuum which is applied to the fetal scalp, this has the consequence of stabilizing and securely fixing the head of the fetus to the suction cup.

The graduated vacuum indicator included in the handle gives a precise indication of the vacuum applied to the fetal scalp. The flexible traction cable allows freedom of movement and orientation for effective traction at the rate of maternal contractions.

# OBSTETRIC SUCTION CUP



## Features

- The device includes a pull handle associated with the vacuum pump and a colour-coded graduated vacuum indicator for ergonomic and efficient use.
- A push button inserted in the handle allows the vacuum to be released easily.
- The suction cup is fitted with a tunnel where the traction pipe is housed for easy insertion and positioning of the device.
- The vacuum hose contains two steel traction cables and is fixed in the centre of the suction cup. Its flexibility allows freedom of movement and a 180 ° traction angle. It also has two marks (6 and 11cm) which help to assess the insertion distance and the presentation height.
- Latex-free system, DEHP-free (Di-2-ethylhexyl phthalate), BPA-free (bisphenol A).



## Reference and packaging

- Product reference: HK-TTQ-A, HK-TTQ-B
- Packaging: 5 units / box. Each suction cup system is contained in an individual sterile bag.

## Product Enquiries/Ordering



Beijing HangTian KaDi Technology R&D Institute  
3th Floor, Building of 9th, 1 Area, North of Liandong U Zone, Majuqiao Town, Tongzhou  
District, Beijing, China  
Tel: +86 13910611697 Email: jeffjiang@bj-htkd.com

CE 0123 Class IIa

Read the instructions on the package leaflet carefully. Must be used by qualified and trained healthcare professionals.