




# Full Face Mask


## Ease Fit FMI

*Make breath naturally happen!*

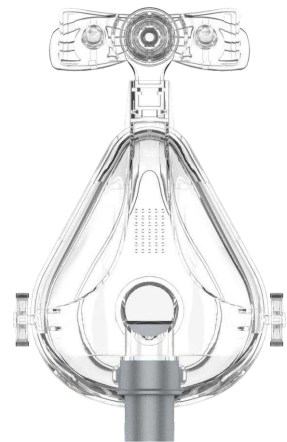


### Forehead stepless knob

- Independent innovation;
- Stepless knob design to fit forehead easily;
- Perfectly suit individual facial contour.

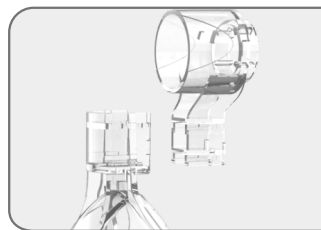

### Adjustable height

- 2 levels of height are available.

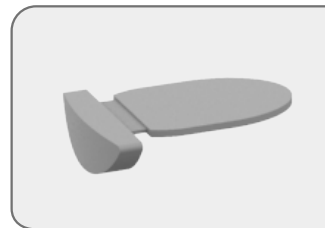

### Safety valve

- Anti-asphyxia safe design;
- Ensure greater sense of safety;




### Double cushions

- Medical liquid silicon;
- A comfortable seal is achieved by dual-wall design;
- Easy to clean and reassemble;



### Arc-buckle

- Curve buckle structure enables flexible rotation;
- Easy to attach, easy to adjust;

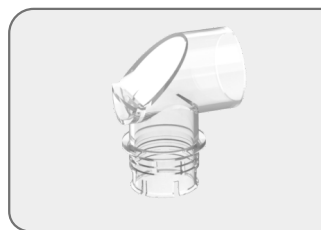


### 360° Rotary joint

- Flexible tubing positions are fulfilled;
- Free to change sleeping positions.



## Dimension Details

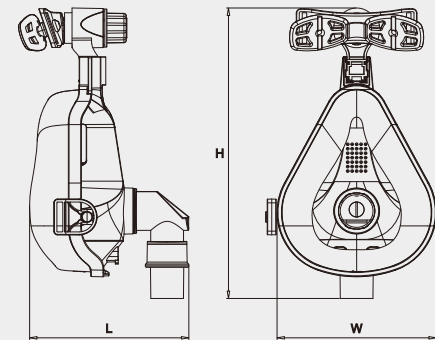

Dimension Size	H(mm)	W(mm)	L(mm)
Large	194.9	122	114.5
Medium	183.9	118	114.5
Small	175.9	110	112.5



## Wearing Instructions




1 Put the headgear to an "inverted triangle" shape with the black side up; and then fasten the upper parts through the buckles on the top of the mask frame;



2 Rightly wear the mask on head and fix it;



3 Tie up the lower parts of the headgear to other two buckles;

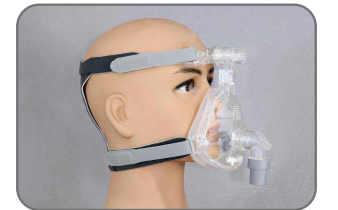

4 Adjust the position of headgear and forehead pad to make the mask perfectly fit.



## Cleaning Guide

### Daily Use

Slightly wash the mask accessories (except for headgear) with soap in warm water; after cleaning, dry them in natural way. Use a soft brush to clean the vent hole.

### Weekly Washing

Clean the headgear with hand. First soak it in warm soap water for less than 10 min; and then dry it in natural way. Avoid expose it to direct sunlight.



## Accessory Replacement

**Silicon cushion** >>> Every 6 months | **Headgear** >>> Every 6 months | **Mask frame** >>> Every 12-18 months

**24 hour service hotline** // +86 18674382266 (WeChat & WhatsApp)

Email: sale@csbeyond.com URL: www.hnbyond.com

Add: Beyond Zone, Lijiacun Rd, Xueshi Street, Yuelu District, 410208, Changsha, China

