




# CPAP/BIPAP Masks

## EaseFit FMIIP

*Make breath naturally happen!*

- Safe and comfortable;
- Three sizes are available;
- Multiple innovation patents;
- Suitable for various non-invasive ventilator.

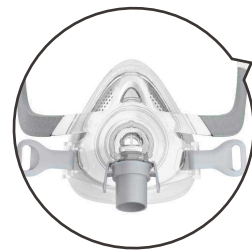


### Flexible and Lightweight

- Lightweight frame design, no forehead support, reduces eye occlusion, give wider field of vision.



### Soft and Comfortable

- High-end Double Bubbles liquid silicone technology, softly fits the face, prevents air leakage, and enjoys comfortable breathing.
- Adaptive pleated silicone cushion, automatically adjust to fit face, relieve the pressure on bridge of nose.

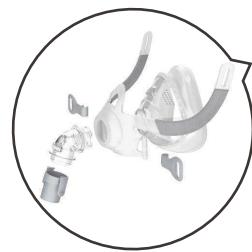

### Easy to use


- Headgear buckle can be quickly removed for easy wearing and removal.
- Headgear buckle can be quickly removed for easy wearing and removal.



### Safe to use

- Medical grade odorless silica gel, safe material, feel free to use.
- Soft and skin-friendly headgear, passed BIOC test, suitable for sensitive skin.


## Dimension Details

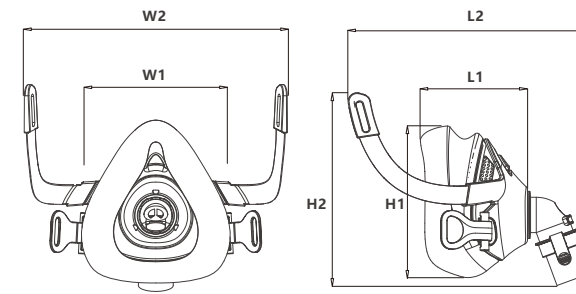

Dimension Size	W1(mm)	W2(mm)	H1(mm)	H2(mm)	L1(mm)	L2(mm)
Large	95	166.4	114.6	130.8	73	162.5
Medium	95	166.4	102.9	130.8	72.7	162.5
Small	95	166.4	93.8	130.8	71.7	162.5

Note: allowable tolerance ± 10mm

## Wearing Instructions

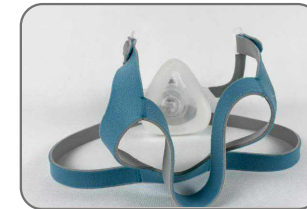

»1 Put the headgear to an "inverted triangle" shape with the black side up; and then fasten the upper parts through the buckles on the top of the mask frame;



»2 Rightly wear the mask on head and fix it;



»3 Tie up the lower parts of the headgear to other two buckles;



»4 Adjust the position of headgear and face bracket to make the mask perfectly fit.

## Cleaning Guide

### Daily Use

Slightly wash the mask accessories (except for headgear) with soap in warm water; after cleaning, dry them in natural way. Use a soft brush to clean the vent hole.

### Weekly Washing

Clean the headgear with hand. First soak it in warm soap water for less than 10 min; and then dry it in natural way. Avoid expose it to direct sunlight.

## Accessory Replacement

**Silicon cushion** »» Every 6 months | **Headgear** »» Every 6 months | **Mask frame** »» Every 12-18 months

**24 hour service hotline** // +86 18674382266 (WeChat & WhatsApp)

Email: sale@csbeyond.com URL: www.hnbyond.com

Add: Beyond Zone, Lijiacun Rd, Xueshi Street, Yuelu District, 410208, Changsha, China

