


# Oxycel Powder

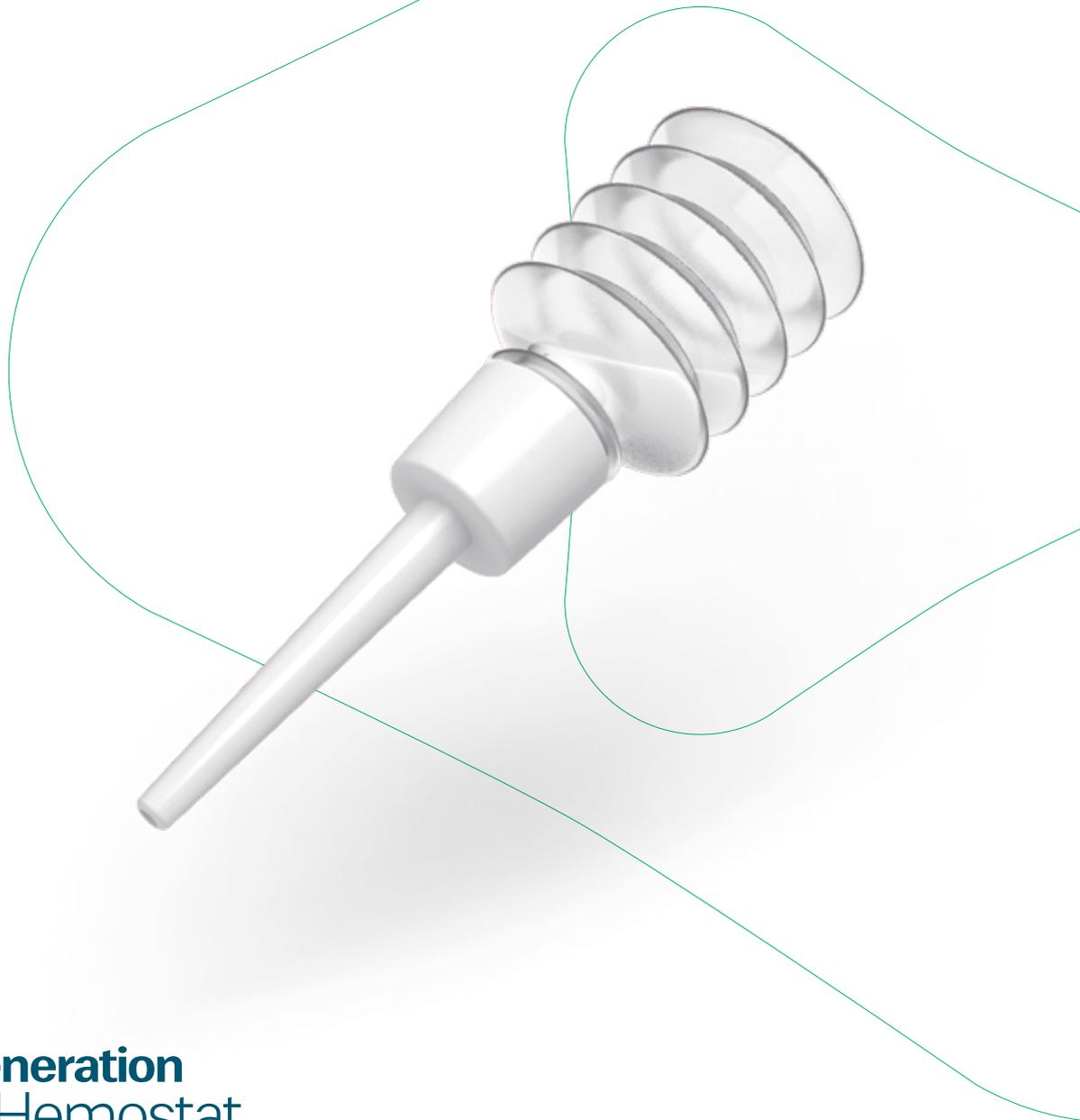

**Latest Generation**  
Powder Hemostat



# Oxigel Powder

The new generation of topical absorbable hemostats, Oxigel Powder, has been developed to control persistent oozing bleeding on large surfaces. The particles of Oxigel Powder are biocompatible, non-pyrogenic and derived from purified plant starch, so there are no human or animal materials.

## Application Technique

### 1 DRY

Gauze dry the wound and remove all excess blood.

### 2 APPLY

Immediately after removing excess blood, apply OXIGEL POWDER over the entire wound.

### 3 COMPRESS

Apply direct pressure over OXIGEL POWDER using moist gauze for 1 to 2 minutes.

### 4 IRRIGATE

Remove excess OXIGEL POWDER after hemostasis is achieved.

## Immediate Hemostasis Absorption within hours

Oxigel Powder upon contact with blood, initiates the clotting process. Blood solids such as platelets, red blood cells and blood proteins are concentrated in powder particles to create a gelled matrix. The normal clotting mechanism is improved, regardless of the coagulation status of the patient, by creating a barrier to further blood loss. Oxigel Powder absorbed within 24-48 hours.

## Proven protection and effectiveness in a range of surgical areas and types of procedures

- General surgery
- Gynecological
- Urology
- Cardiac and vascular surgery
- Neurosurgery
- Traumatology
- Orthopedicsurgery
- Plasticsurgery
- ENTsurgery



## Safe

- 100% plant-based polysaccharides
- Naturally Safe
- Biocompatible
- Absorbable in 24-48 hours

## Simple


- No additional prep time necessary
- No special storage conditions required
- No mixing or refrigeration
- Easy to use applicators

## Effective

- Complete hemostasis in minutes
- Ensure wide area coverage on rough surfaces and in areas hard to reach
- Cost Effective


## Delivery System:

Applying Oxigel Powder correctly and directly is quick and easy with the spray applicator. A light, plastic device with a long, flexible tube is featured in the distribution system. Extended reach applicator tips from Oxigel Powder Applicators provide accurate and precise delivery of Oxigel Powder hemostatic powder from a simple, single-use device.

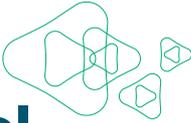

# Oxigel Powder

## OPEN SURGERY AND LAPAROSCOPIC APPLICATORS

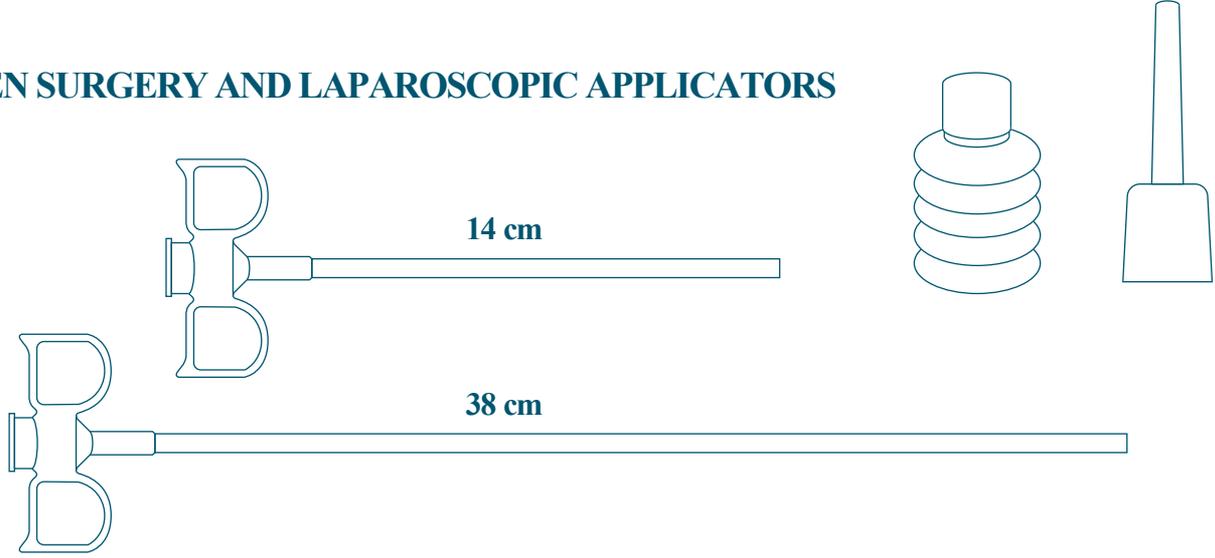

### Ordering Information

| Code       | Oxigel Powder | Applicator | Units Per Box |
|------------|---------------|------------|---------------|
| OXPO1      | 1 gr          | -          | 5 per box     |
| OXPO2      | 2 gr          | -          | 5 per box     |
| OXPO3      | 3 gr          | -          | 5 per box     |
| OXPO4      | 4 gr          | -          | 5 per box     |
| OXPO5      | 5 gr          | -          | 5 per box     |
| OXPO1-EA14 | 1 gr          | 14 cm      | 1 set/box     |
| OXPO2-EA14 | 2 gr          | 14 cm      | 1 set/box     |
| OXPO3-EA14 | 3 gr          | 14 cm      | 1 set/box     |
| OXPO4-EA14 | 4 gr          | 14 cm      | 1 set/box     |
| OXPO5-EA14 | 5 gr          | 14 cm      | 1 set/box     |
| OXPO1-EA38 | 1 gr          | 38 cm      | 1 set/box     |
| OXPO2-EA38 | 2 gr          | 38 cm      | 1 set/box     |
| OXPO3-EA38 | 3 gr          | 38 cm      | 1 set/box     |
| OXPO4-EA38 | 4 gr          | 38 cm      | 1 set/box     |
| OXPO5-EA38 | 5 gr          | 38 cm      | 1 set/box     |
| EA14       | -             | 14 cm      | 5 per box     |
| EA38       | -             | 38 cm      | 5 per box     |

**Betatech Medikal Cihazlar  
San. Müm. İç ve Dış  
Tic. Ltd.Şti.**

İkitelli Org. San. Bölgesi  
Atatürk Oto San. Sitesi  
22. Sk. Ünal İş Merkezi No: 9,  
Başakşehir-İstanbul, TÜRKİYE

**T.+90 212 671 0683**

**F.+90 212 671 0684**

**E. info@betatechmedical.com**

**BETATECH  
MEDICAL.COM**

